



Oracol[®] spray

Medical device

15 ml bottle with oral spray dispenser

Oracol[®] spray is a medical device useful for the protection and repair of the mucosa of the oral cavity affected by trauma and irritations of various origin. It contains carrageenan, which creates an invisible film on the surface of mucosa and gums, isolating it from the external environment and so favouring the physiological tissue regeneration process.

INDICATIONS

Useful in case of irritations and inflammations of the oral cavity and gums. Helps in providing relief from pain caused by oral mucosal lesions of various origins: stomatitis, gingivitis, mouth ulcers, trauma.

Gluten-free

INSTRUCTIONS FOR USE

Remove the protective cap from the nozzle and direct it towards the area to be treated inside the mouth. Spray the product 2 or 3 times onto the irritated oral mucosa to create a protective film and repeat the treatment twice a day (or more) as needed, preferably in the morning and evening, after meals and following thorough teeth cleaning. Continue treatment for at least 2 weeks or until symptoms disappear.

WARNINGS

Harmless if accidentally ingested. **Keep out of the reach of children.**

Use of the spray is strictly personal. Use the product within 15 days after first opening.

PRECAUTIONS

After each use it is recommended not to eat or drink for about 60 minutes, to prevent an early removal of the product.

CONTRAINDICATIONS

Biocompatibility studies have not revealed any contraindications or undesirable effects; however, use of the product should be avoided in patients with a history of hypersensitivity to any of its components.

If in doubt, consult your doctor before using the product.

SIDE EFFECTS

No undesirable side-effects have been reported. Inform directly your doctor or pharmacist if you experience any side effects attributable to use of the product.

STORAGE CONDITIONS

Store at room temperature, away from heat sources. The expiry date refers to the unopened product, correctly stored. Do not use the product after its expiry date. Make sure to replace the cap on the dispenser after each use. **Do not dispose of in the environment after use.**

Instructions for use last reviewed: 13 July 2018



» Scan the QR Code with your smartphone



» Explore the entire Nóos product line



Gastrointestinal



Respiratory



Immunity



Hearing



Oral cavity



Eye care



Rare diseases

 **Manufacturer:** Difa Cooper Spa
Via Milano 160 - 21042 Caronno Pertusella (VA) Italy

Distributed by:



Via Campello sul Clitunno, 34 - 00181 Roma
www.noosit.com



Check your local municipal guidelines