

Proctoglu[®] plus

Medical device

Proctologic cream

INDICATIONS

Proctoglu[®] plus is a medical device used to relieve symptoms associated with haemorrhoidal disorder, fissures and proctitis (burning, itching, bleeding, pain). The application of Proctoglu[®] plus is useful to reduce friction during excretion, and alleviate burning, itching and bleeding. It also prevents inflammation and congestion, reduces pain and protects the damaged mucous membrane of the rectum, facilitating healing.

HOW IT WORKS

Proctoglu[®] plus reduces friction during excretion, thanks to its lubricating and moisturizing properties. This facilitates the expulsion of faeces and protects the inflamed and irritated anorectal surface. It helps prevent irritation and congestion of the perianal area, and provides relief from pain, burning and itching.

INGREDIENTS

Vaseline, purified water, vaseline oil, cetostearyl alcohol, stearyl alcohol, cetyl alcohol, glycerin, chamomile hydroglycerin extract, phenoxyethanol, beta-glucan, allantoin, methylparaben, ethylparaben, lactic acid, sodium hydrate.

PACKAGING

Proctoglu[®] plus is available in a 30 ml tube supplied with a rectal cannula.

The cannula is a  medical device manufactured by INGE S.p.A. - Via Garibaldi 190, Garbagnate Milanese (MI) Italy

INSTRUCTIONS FOR USE

Apply the cream 2 or 4 times a day, preferably before and after each evacuation. Wash hands before and after each application.

Before applying the product, clean the affected area. When using for the first time, remove the cap and turn it over. Press to pierce the aluminium seal in the mouth of the tube. Take a small amount of cream and apply it to the perianal area with your fingers.

Attach the applicator to the tip of the tube. Press the tube until a small amount of cream can be seen coming out of the cannula holes. Use this to lubricate the cannula by spreading it over its surface. Insert the applicator into the anus.

Press gently on the end of the tube to apply the cream to the affected area. Extract the applicator.

After use, immediately remove the cannula from the tube and replace the cap. Thoroughly wash the cannula with water and cover it with its case.

STORAGE

Keep out of the sight and reach of children. Store the product in a cool, dry place, away from light and heat sources. The expiry date refers to the product in unopened packaging and correctly stored. After opening, use the product within 3 months.

WARNINGS

Do not use after the expiry date indicated on the packaging. Do not use if the packaging has been opened or damaged. The use of Proctoglu® plus and the cannula are strictly personal.

Do not use in children under 12 years of age. Consult your doctor for use during pregnancy or breastfeeding.

Seek your doctor's advice before using Proctoglu® plus, in case it is necessary to apply other products for endorectal or perianal use. Do not use the cream in the case of allergy or hypersensitivity to one or more ingredients. In the event of undesirable effects, discontinue treatment and consult your doctor. If there is no improvement after 2 weeks of treatment, consult your doctor.

» Scan the QR Code with your smartphone



» Explore the entire Nóos product line



Gastrointestinal

Respiratory

Immunity

Hearing

Oral cavity

Eye health

Rare diseases

Date of last review: January 2020
(IFU1_01/20r0_07/21)

Medical device  0373

 **Manufacturer:** Biofarma
Via Castelliere 2 - Mereto di Tomba (UD)

Distributed by:



Via Campello sul Clitunno, 34 - 00181 Rome
www.noosif.com

NÓOS FOR THE ENVIRONMENT

PAP 22

LEAFLET

PAPER

Check your local regulations